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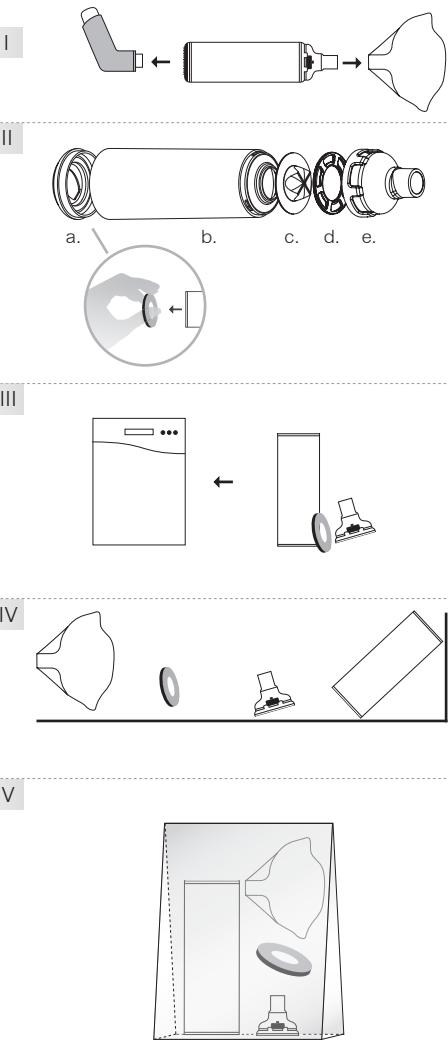
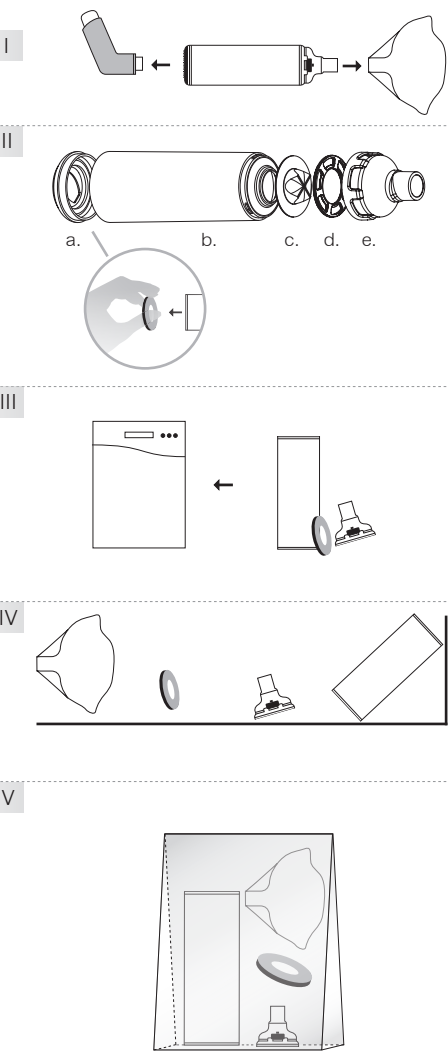
CLINICAL REPROCESSING

Cleaning / Disinfection

I Disconnect the metered dose inhaler (MDI). Optionally, when using a mask: remove the mask from the delivery chamber.

II Disassemble the RC-Chamber® reusable completely into it's 5 parts as shown in figure II. First remove the transparent plastic ring (a.) with the fixed integrated rubber adapter by inserting your thumb deep into the opening. Pull out the complete back piece with force. Afterwards disassemble the mouthpiece into the several parts (a.– e.).

III Place all the individual parts in the hospital dishwasher and select the cleaning program with 93° C (exposure time 5 minutes). Please observe the manufacturer's instructions. Use a pH-neutral detergent and observe the dosage recommendations by the manufacturer.

IV Lay the individual parts in their open position to air-dry. Ensure sufficient drying time. Do not rub them dry!

Sterilization

V Sterilization is possible via the following procedures: Place all disassembled parts into a suitable sterilization package. Ensure that all the parts are dry. Sterilize the RC-Chamber® reusable according to the steam sterilization process (DIN EN ISO 17665) using the following temperatures:

- max. 134 °C (reaction time at least 3 minutes)
- 121 °C (reaction time 15 minutes)

The recommended cooling time is 30 minutes.

Reassemble all the pieces according to figure II. Make sure that the mouthpiece is locked in place with a “click.”

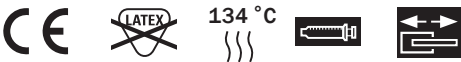
Disassembling the RC-Chamber® reusable results in several pieces which can be swallowed by children. There fore the individual pieces are always to be kept out of reach of children.

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Hersteller:
Medical Developments International (MDI)
4 Caribbean Drive,
Scoresby, Victoria 3179,
Australia

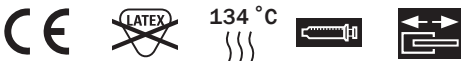
EC REP
EU-Bevollmächtigter:
EMERGO EUROPE
Prinsessegracht 20
NL 2514 AP, The Hague

DISPOSAL

Dispose the RC-Chamber® reusable appropriately at the end of its product life.

MATERIAL DURABILITY

RC-Chamber® reusable can be reprocessed (sterilised) up to 40 times if no rinsing liquid is used. The RC-Chamber® reusable as well as the mask should be replaced after 12 months.

In some instances, product graphics may vary from actual product models, sizes and /or colors listed below.

For home care we recommend:

The RC-Chamber® models (with and without mask) optimize inhalation with MDIs, including emergency plan and guardian angel pendant.

3007a - 11/2020

CEGLA MEDIZINTECHNIK

Exclusive Distribution Germany, Austria and Switzerland.
Manufacturer mask

CEGLA Medizintechnik GmbH & Co. KG
Horresser Berg 1
56410 Montabaur
Germany
Tel +49 2602 92 13-0
Fax +49 2602 92 13-15
www.cegla.de

Manufacturer:
Medical Developments International (MDI)
4 Caribbean Drive,
Scoresby, Victoria 3179,
Australia

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RC-CHAMBER® REUSABLE

For hospital use

Please read the RC-Chamber® reusable instructions for use carefully before first use. Please also follow the instructions of the nebulizer.

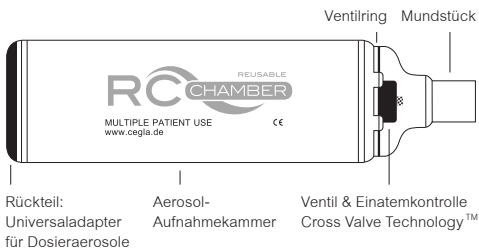
The RC-Chamber® reusable package contains:

- Mouthpiece
- Valve & inhalation control Cross Valve Technology™
- Spray entrance chamber
- Back piece: transparent plastic ring with permanently integrated rubber adapter
- The items 3071, 3081 and 3091 each include a mask

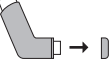
► A detailed clinical reprocessing instruction is available at info@cegla.de

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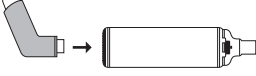
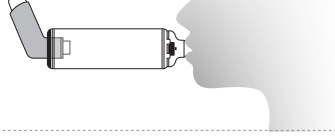
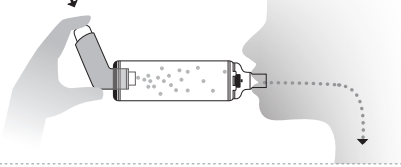
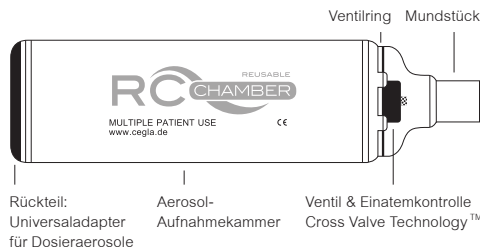
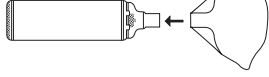
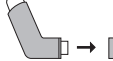
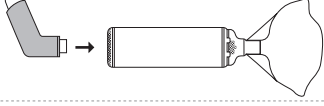
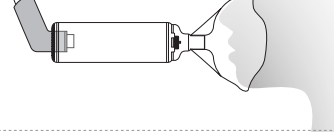
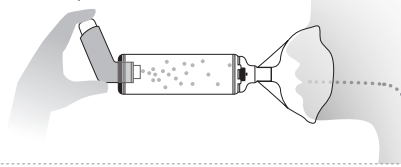
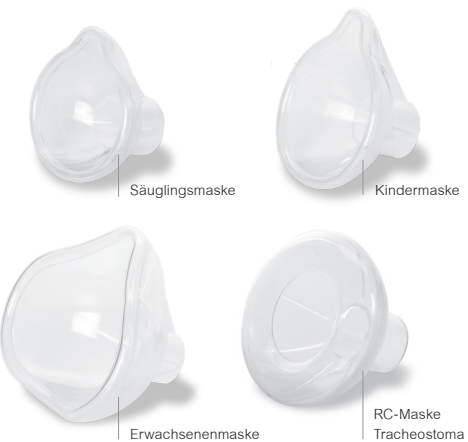
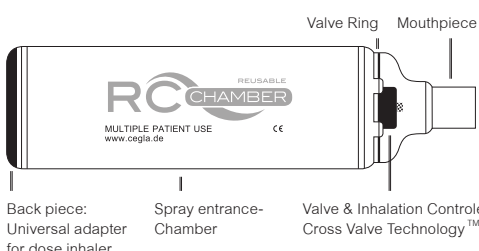
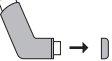
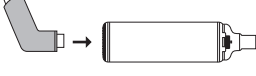
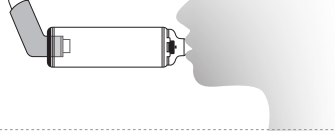
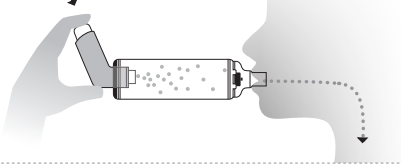
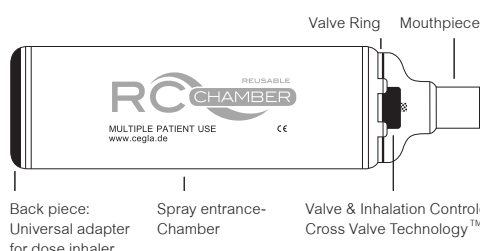
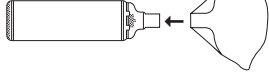
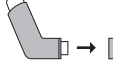
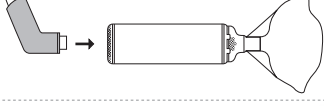
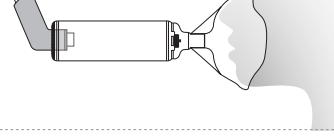
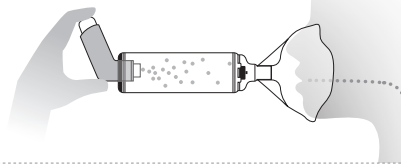
VII

INSTRUCTIONS FOR USE

I Continually check for foreign objects in the RC-Chamber® reusable, for damage to any of the parts or if any parts are missing.

II For mask use: Place the mask on the RC-Chamber® reusable mouthpiece.

III Remove the sealing cap from the mouthpiece of the dose inhaler and shake dose inhaler spray vigorously before each use.

IV Place the mouthpiece of the dose inhaler as shown in the figure in the provided rubber adapter of the RC-Chamber® reusable.

V Place your lips tightly around the mouthpiece or place the mask airtight over the nose and mouth.

VI Push the cartridge of the dose inhaler down in order to dispense a spray into the chamber of the RC-Chamber® reusable.

VII Breathe in and hold the breath for 5 to 10 seconds to optimize the effect. Children under 18 months should inhale 4 to 6 breaths, children and adults 2 to 4 breaths. Should several sprays be prescribed, then a one minute break should be taken before repeating steps V to VII!

PURPOSE FOR USE

The RC-Chamber® reusable is an inhalation device which was especially developed for MDI therapy with metered dose inhalers – so-called dose inhaler – and can be used for treatment of respiratory illnesses. The flexible rubber adapter is compatible with all common dose inhaler.

With the help of the RC-Chamber® reusable the coordination between releasing the dose inhaler and inhalation of the medicine is omitted. In addition the large, not respirable particles are filtered out. Therefore irritation as well as a fungal attack in the mouth and throat is prevented.

AVAILABLE PRODUCTS FOR THE CLINIC

- RC-Chamber® reusable**
- With mouthpiece: article-no. 3061
 - With infant mask: article-no. 3071
 - With child mask: article-no. 3081
 - With mask for adults as well as older children: article-no. 3091
- Infant mask:** article-no. 30110
- Child mask:** article-no. 30210
- Mask for adults and older children:** article-no. 30410
- RC-Tracheal mask:** article-no. 3012