355043A

Urgosterile

pansement stérile adhésif

FR

COMPOSITION :

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EN

COMPOSITION:

The **Urgostérile** dressing is a sterile adhesive dressing consisting of an extensible white non-woven fabric backing, coated with an adhesive mass, and a non-woven integrated and absorbent compress covered with an anti-adhesive layer. Sterile dressing (sterilized with ethylene oxide).

INSTRUCTIONS FOR USE:

1. Check the integrity of the pouch before use.
2. Carefully clean the part to be protected and its periphery.
3. Remove the dressing from its packaging on the side indicated and remove the protective tabs.
4. Apply the dressing so that the central pad covers the wound.
5. Adhere the dressing well on all 4 sides.
6. Avoid tension during application.
7. Renew the dressing as often as necessary.

PROPERTIES:

- Micro-aerated, it lets the wound breathe and avoids maceration.
- Optimal hold, leaves no residue on the skin.
- Sterile dressing reducing the risk of infection.
- With high cutaneous tolerance, it respects the skin
- The compress absorbs exudates without adhering to the wound thanks to its anti-adhesive layer.
- Adhesive on its 4 sides, it totally protects the wound

INDICATIONS:

Protection and coverage of superficial wounds (abrasions, cuts), low-exuding sutured wounds and post-operative wounds.

A wound or a cut is considered as superficial if it only reaches the skin or the immediate underlying fats. If an accidental superficial wound overpasses 3 cm, a doctor should be consulted within the 6 hours following the accident to determine if the wound should be sutured or cared (glued with a special surgical adhesive, ...)

PRECAUTIONS FOR USE :

For people under 16 years, it must be applied by an adult. Single-use sterile dressing, do not re-use or re-position a dressing already applied: risk of infection and reduced effectiveness in fixing and maintaining of the dressing.

Do not re-sterilise. Check the integrity of the sterile protector before use. Do not use if the package is damaged.

Inspect the wound and the surrounding area regularly in order to detect any signs of infection such as pain, redness, oedema, odor or pus formation.

Do not use on deep non-sutured wounds, on bleeding or highly exudative wounds.

Any serious incident linked to the product has to be reported to the manufacturer and the National Competent Authority.

Method of destruction: by incineration. If incineration is not possible, waste disposal is carried out in accordance with the environmental procedures in force.

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