US

PRODUCT DESCRIPTION
Single use hydrophilic urinary catheter.

MATERIAL
Hydrophilic coated urinary catheter. Polyolefin-based elastomer (POBE) with PVP and NaCl coating.

INDICATIONS FOR USE
Intermittent urinary catheterization.

PATIENT POPULATION
For adults, adolescents, children, and infants.

WARNINGS AND PRECAUTIONS
For single use only.
Once used the surface coating will deteriorate and is no longer sterile.
Reuse may lead to discomfort, urethral damage, or infection.
Do not use a product if the sterile packaging is broken or damaged. Urinary catheterization therapy is associated with an increased risk of urethral bleeding, trauma and/or infection.
LoFric catheters are for prescription use only.
Follow instructions and advice from your healthcare professional. Contact your prescriber if you experience difficulties.

ADVERSE REACTIONS
Common adverse reactions (>1/100) related to catheterization therapy includes urethral damage and urinary tract infection. If unexpected discomfort, sign of trauma or infection occurs, discontinue use and consult your prescriber.

Any serious adverse reaction occurring when using the catheter should be reported to the manufacturer and your local health authority.

PACKAGE AND STORAGE
Store in their package in a dry place, at room temperature. Use before expiry date on package.

UK/CA/AU

PRODUCT DESCRIPTION
Single use hydrophilic urinary catheter.

MATERIAL
Hydrophilic coated urinary catheter. Polyolefin-based elastomer (POBE) with PVP and NaCl coating.

INDICATIONS FOR USE
For short and long term intermittent urinary catheterization.

PATIENT POPULATION
For adults, adolescents, children, and infants.

WARNINGS AND PRECAUTIONS
For single use only.
Once used the surface coating will deteriorate and is no longer sterile.
Reuse may lead to discomfort, urethral damage, or infection.
Do not use a product if the sterile packaging is broken or damaged. Urinary catheterization therapy is associated with an increased risk of urethral bleeding, trauma and/or infection.
LoFric catheters are for prescription use only.
AU: Always consult a healthcare professional before using LoFric catheters.
Follow instructions and advice from your healthcare professional. Contact your prescriber if you experience difficulties.

ADVERSE REACTIONS
Common adverse reactions (>1/100) related to catheterization therapy includes urethral damage and urinary tract infection. If unexpected discomfort, sign of trauma or infection occurs, discontinue use and consult your prescriber.

Any serious adverse reaction occurring when using the catheter should be reported to the manufacturer and your local health authority.

PACKAGE AND STORAGE
Store in their package in a dry place, at room temperature. Use before expiry date on package.

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UK:
For more information: www.wellspect.co.uk, or call us at (0)1453 791763.

US:
For more information please visit www.wellspect.us, or call us toll free at 877-4-LOFRIC (456-3742).

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OTHER MARKETS:
For further information visit www.wellspect.com

SYMBOLS GLOSSARY:

Do not use if package is damaged

Keep away from direct sunlight

Consult instructions for use

Single use

Sterilized using ethylene oxide

Use by date

Manufacturer

Manufacturing date

Rx Only
For prescription use only (US)









A	15cm/6"	20cm/8"	30cm/12"	40cm/16"
B	12cm/5"	18cm/7"	27cm/10,5"	35cm/14"

Sterile barrier
Sterile Versiegelung
Barrière stérile
Steriele barrière
Barriera sterile
Barrera estéril



Medical device
Medizinprodukt
Dispositif médical
Medisch hulpmiddel
Dispositivo medico
Producto sanitario

Wellspect HealthCare
Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden.
Tel: +46 31 376 40 00. www.wellspect.com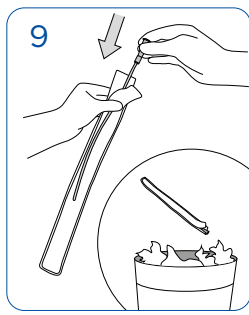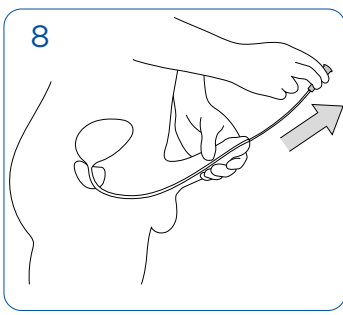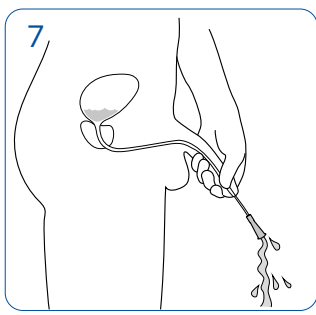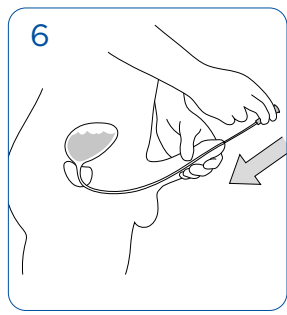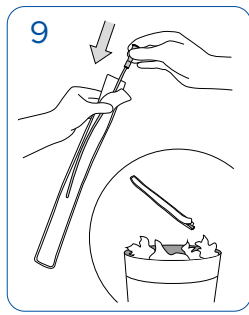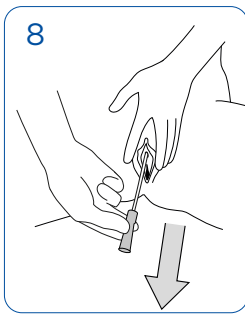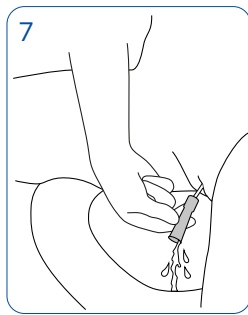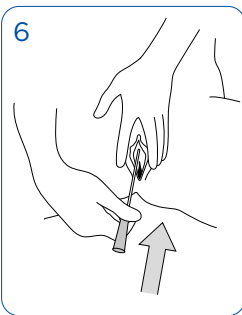

Tiemann/Coudé

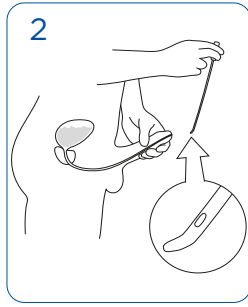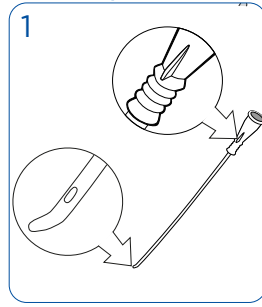US

INSTRUCTIONS FOR USE

1. Wash your hands with soap and water before catheterization.
2. To open, peel the tabs on the funnel side of the package. Pull the tabs open about 2 inches.
3. Fill the package with water. Use sterile water only as directed by your healthcare provider. Soak the catheter for 30 seconds before use.
4. Whilst preparing yourself for catheterization, you can remove the sticker and use the self-adhesive tape to attach the product ta a dry surface.
5. Take out the catheter.

FEMALE

6. Spread the labia and locate the urethra just above the vaginal opening. With the other hand, insert the catheter slowly into the urethra.
7. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
8. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter completely.
9. Put the catheter back in the package and dispose appropriately (local regulations may vary).

MALE

6. Lift the penis towards the stomach to straighten the urethra. Slowly insert the catheter into the urethra without touching the tube. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
7. Angle the penis down as urine begins to flow through the catheter.
8. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter.
9. Put the catheter back in the package and dispose appropriately (local regulations may vary).

Special instruction for Tiemann/Coudé catheter, with slightly curved tip.

- A special technique is required when using a Tiemann/Coudé.
- Speak to your healthcare professional for training and advice.
1. Note where the marker on the funnel is in relation to the curved catheter tip before inserting the catheter. It will guide you keeping the curved tip in the right direction during use.
 2. Keep the curved tip upwards towards the stomach during insertion and throughout catheterization, including withdrawal. Or follow specific instructions given by your healthcare professional.

IMPORTANT INFORMATION

Do not place the catheter on paper or cloth as these materials can absorb the wet coating.

UK/CA/AU

INSTRUCTIONS FOR USE

1. Wash your hands with soap and water before catheterization.
2. To open, peel the tabs on the funnel side of the package.
3. Fill the package with water, at home from the cold tap and in hospital with sterile water or saline. Soak the catheter for 30 seconds before use.
4. Whilst preparing yourself for catheterization, you can remove the sticker and use the self-adhesive tape to attach the product ta a dry surface.
5. Take out the catheter.

FEMALE

6. Spread the labia and locate the urethra just above the vaginal opening. With the other hand, insert the catheter slowly into the urethra.
7. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
8. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter completely.
9. Put the catheter back in the package and dispose appropriately (local regulations may vary).

MALE

6. Lift the penis towards the stomach to straighten the urethra. Slowly insert the catheter into the urethra without touching the tube. When urine begins to flow, insert the catheter slightly more to ensure both eyelets are inside the bladder.
7. Angle the penis down as urine begins to flow through the catheter.
8. When the urine flow stops, slowly withdraw the catheter a little bit. If urine starts to flow again, wait until it has stopped to ensure complete bladder emptying. Then remove the catheter.
9. Put the catheter back in the package and dispose appropriately (local regulations may vary).

Special instruction for Tiemann/Coudé catheter, with slightly curved tip.

- A special technique is required when using a Tiemann/Coudé.
- Speak to your healthcare professional for training and advice.
1. Note where the marker on the funnel is in relation to the curved catheter tip before inserting the catheter. It will guide you keeping the curved tip in the right direction during use.
 2. Keep the curved tip upwards towards the stomach during insertion and throughout catheterization, including withdrawal. Or follow specific instructions given by your healthcare professional.

IMPORTANT INFORMATION

Do not place the catheter on paper or cloth as these materials can absorb the wet coating.

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