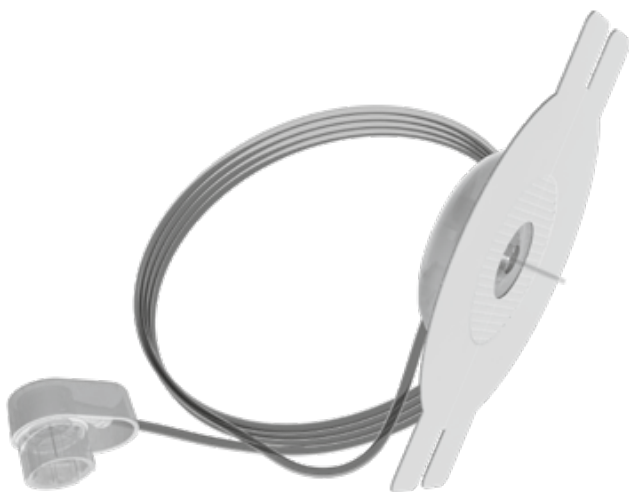
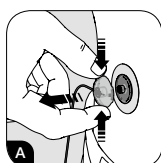
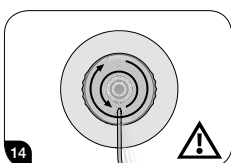
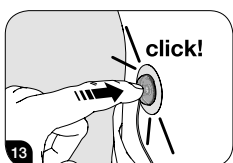
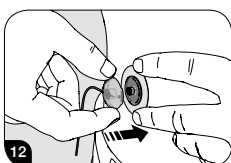
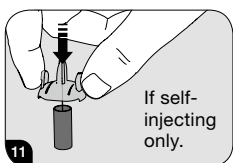
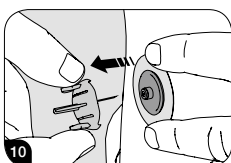
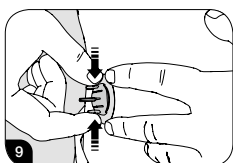
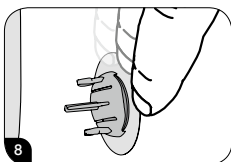
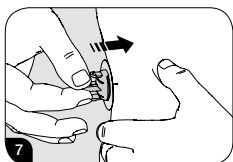
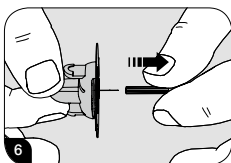
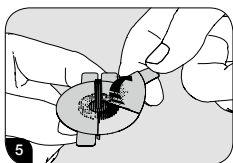
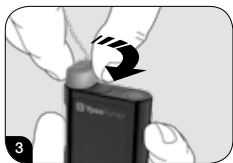
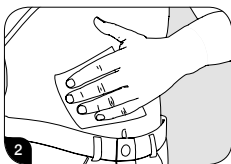
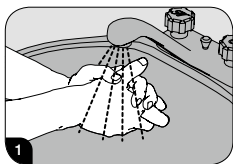
YpsoPump®
Orbit®soft

YpsoPump® Infusion Set

en	Instructions for use	10
ar	إرشادات الاستخدام	17
bg	Инструкции за употреба	18
cs	Návod k použití	22
da	Brugsanvisning	26
de	Gebrauchsanweisung	30
el	Οδηγίες χρήσης	34
es	Instrucciones de uso	38
et	Kasutusjuhend	42
fi	Käyttöohje	46
fr	Mode d'emploi	50
he	הוראות שימוש	57
hr	Upute za uporabu	58
hu	Használati útmutató	62
it	Istruzioni per l'uso	66
lt	Naudojimo instrukcija	70
lv	Lietošanas instrukcija	74
nl	Gebruiksaanwijzing	78
no	Bruksanvisning	82
pl	Instrukcja obsługi	86
pt	Instruções de utilização	90
ro	Instrucțiuni de utilizare	94
ru	Инструкция по использованию	98
sk	Návod na použitie	102
sl	Navodila za uporabo	106
sv	Bruksanvisning	110
tr	Kullanım talimatları	114
zh	使用说明	118

STERILE EO

en	Sterilized using ethylene oxide
ar	مُعقمة باستخدام أكسيد الإيثيلين
bg	Стерилизирано с етиленов оксид
cs	Sterilizováno ethylenoxidem
da	Steriliseret med etylenoxid
de	Sterilisiert mit Ethylenoxid
el	Αποστειρωμένο με αιθυλενοξείδιο
es	Esterilizado con óxido de etileno
et	Steriliseeritud etüleenoksiidiga
fi	Steriloitu etyleenioksidilla
fr	Stérilisé avec de l'oxyde d'éthylène
he	מעוקר באמצעות אתילן-אוקסיד
hr	Sterilizirano etilen oksidom
hu	Etilén-oxiddal sterilizálva
it	Sterilizzato mediante ossido di etilene
lt	Sterilizuota naudojant etileno oksidą
lv	Sterilizēts ar etilēna oksīdu
nl	Gesteriliseerd met behulp van ethyleenoxide
no	Sterilisert med etylenoksid
pl	Wysterylizowany tlenkiem etylenu
pt	Esterilizado com óxido de etileno
ro	Sterilizat cu etilenoxid
ru	Стерилизовано этиленоксидом
sk	Sterilizované etylénoksidom
sl	Sterilizirano z etilenoksidom
sv	Steriliserad med etylenoxid
tr	Etilen oksit ile sterilize edilmiştir
zh	使用环氧乙烷消毒

en	Single sterile barrier system. Sterilized using ethylene oxide.
ar	نظام الحاجز الواحد المعقم. معقم باستخدام أكسيد الإيثيلين.
bg	Система с единична стерилна бариера. Стерилизирано с етиленов оксид.
cs	Systém jednovrstvé sterilní bariéry. Sterilizováno ethylenoxidem.
da	Enkelt steril barrieresystem. Steriliseret med etylenoxid.
de	Sterilbarrieresystem. Mit Ethylenoxid sterilisiert.
el	Σύστημα μονού στείρου φραγμού. Αποστειρωμένο με αιθυλενοξείδιο.
es	Sistema de barrera estéril simple. Esterilizado con óxido de etileno.
et	Ühekordne steriilse barjääri süsteem. Steriliseeritud etüleenoksiidiga.
fi	Yksittäinen steriili estojärjestelmä. Steriloitu etyleenioksidilla.
fr	Système de barrière stérile simple. Stérilisé avec de l'oxyde d'éthylène.
he	מערכת חסם סטרילי יחיד. מעוקר באמצעות אתילן אוקסיד.
hr	Sustav s jednostrukom sterilnom barijerom. Sterilizirano etilen oksidom.
hu	Egyszeres steril gátrendszer. Etilén-oxiddal sterilizálva.
it	Sistema di barriera sterile singolo. Sterilizzato mediante ossido di etilene.
lt	Vienguba sterili barjero sistema. Sterilizuota naudojant etileno oksidą.
lv	Vienas sterilas barjeras sistēma. Sterilizēta ar etilēna oksīdu.
nl	Enkel steriel barrieresysteem. Gesteriliseerd met behulp van ethyleenoxide.
no	System med én steril barriere. Sterilisert med etylenoksid.
pl	System pojedynczej bariery sterylnej. Wysterylizowany tlenkiem etylenu.
pt	Sistema de barreira estéril simples. Esterilizado com óxido de etileno.
ro	Sistem cu barieră sterilă unică. Sterilizat cu etilenoxid.
ru	Единая барьерная система стерилизации. Стерилизовано этиленоксидом.
sk	Jednoduchý sterilný bariérový systém. Sterilizované etylénoksidom.
sl	Enojni sterilni pregradni sistem. Sterilizirano z etilenoksidom.
sv	Enkelt steril barrieresystem. Steriliserad med etylenoxid.
tr	Tek steril bariyer sistemi. Etilen oksit ile sterilize edilmiştir.
zh	单次无菌屏障系统。使用环氧乙烷灭菌。

en Consult instructions for use
ar اتبع إرشادات الاستخدام
bg Направете справка с инструкциите за употреба
cs Prostudujte si návod k použití
da Se venligst brugervejledningen
de Gebrauchsanweisung beachten
el Ανατρέξτε στις οδηγίες χρήσης
es Consulte las instrucciones de uso
et Järgige kasutusjuhendit
fi Katso käyttöohjeita
fr Consulter les instructions d'utilisation
he יש לעיין בהוראות השימוש
hr Pogledajte upute za uporabu
hu Lásd a használati utasítást
it Consultare le istruzioni per l'uso
lt Skaitykite naudojimo instrukciją
lv Ievērojiet lietošanas instrukciju
nl Raadpleeg gebruiksaanwijzing
no Følg bruksanvisningen
pl Przestrzegać instrukcji obsługi
pt Consultar as instruções de utilização
ro A se consulta instrucțiunile de utilizare
ru Обратитесь к инструкции по применению
sk Pozrite si návod na používanie
sl Upoštevajte navodila za uporabo
sv Följ bruksanvisningen
tr Kullanım talimatlarını dikkate alın
zh 参见使用说明

en Do not re-use
ar لا تقم بإعادة الاستخدام مرة أخرى
bg Само за еднократна употреба
cs Nepoužívejte opakovaně
da Må ikke genanvendes
de Nicht wiederverwenden
el Μην επαναχρησιμοποιείτε
es No reutilizar
et Ärge kasutage korduvalt
fi Ei saa käyttää uudelleen
fr Ne pas réutiliser
he לא לשימוש חוזר
hr Nije za višekratnu uporabu
hu Ne használja újra
it Non riutilizzare
lt Negalima naudoti pakartotinai
lv Nelietot atkārtoti
nl Niet hergebruiken
no Kun for engangsbruk
pl Nie używać ponownie
pt Não reutilizar
ro A nu se reutiliza
ru Не использовать повторно
sk Nepoužívaťe opakovane
sl Ni za ponovno uporabo
sv Endast för engångsbruk
tr Tekrar kullanmayın
zh 不得重复使用

en Non-pyrogenic
ar لا تؤدي للإصابة بالحمى
bg Непирогенно
cs Nepyrrogení
da Non-pyrogen
de Pyrogenfrei
el Μη πυρετογόνο
es Apirógeno
et Mittepürogeenne
fi Pyrogeeniton
fr Apyrogène
he לא פירוגני
hr Nepirogeno
hu Nem pirogén
it Apirogeno
lt Nepirogeniškas
lv Apirogēns
nl Niet-pyrogeen
no Ikke-pyrogen
pl Niepirogenny
pt Não-piragénico
ro Apirogen
ru Апирогенно
sk Neryrogenne
sl Apyrogeno
sv Pyrogenfri
tr Pirojenik değildir
zh 无热原

en Do not use if package is damaged
ar لا تستخدم المنتج في حالة تلف العبوة
bg Не използвайте, ако опаковката е повредена
cs Nepoužívejte, pokud je obal poškozený
da Må ikke anvendes, hvis emballagen er beskadiget
de Nicht verwenden, wenn Verpackung beschädigt ist
el Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά
es No utilizar si el embalaje está dañado
et Ärge kasutage, kui pakend on kahjustatud
fi Ei saa käyttää, jos pakkaus on vaurioitunut
fr Ne pas utiliser si l'emballage est endommagé
he אין להשתמש אם האריזה ניזוקה
hr Ne koristiti ako je pakiranje oštećeno
hu Ne használja, ha a csomagolás megsérült.
it Non utilizzare se la confezione è danneggiata
lt Nenaudoti, jei pakuotė pažeista
lv Nelietot, ja iepakojums ir bojāts
nl Niet gebruiken als verpakking beschadigd is
no Skal ikke brukes hvis emballasjen er skadet
pl Nie używać, jeśli opakowanie jest uszkodzone
pt Não utilizar se a embalagem estiver danificada
ro A nu se utiliza dacă ambalajul este deteriorat
ru Не использовать при повреждении упаковки
sk Nepoužíte, ak je obal poškodený.
sl Ne uporabljajte, če je ovojnina poškodovana
sv Använd inte produkten om förpackningen är skadad
tr Ambalajı hasarlıysa kullanmayın
zh 若包装破损不得使用

MD

en	Medical device
ar	جهاز طبي
bg	Медицинско изделие
cs	Zdravotnický prostředek
da	Medicinsk udstyr
de	Medizinprodukt
el	Ιατροτεχνολογικό προϊόν
es	Producto sanitario
et	Meditsiiniline seade
fi	Lääkinnällinen laite
fr	Dispositif médical
he	מכשור רפואי
hr	Medicinski proizvod
hu	Gyógyászati eszköz
it	Dispositivo medico
lt	Medicinos prietaisas
lv	Medicīniskā ierīce
nl	Medisch hulpmiddel
no	Medisinsk utstyr
pl	Urządzenie medyczne
pt	Dispositivo médico
ro	Dispozitiv medical
ru	Медицинское изделие
sk	Zdravotnícka pomôcka
sl	Medicinski pripomoček
sv	Medicinteknisk produkt
tr	Tıbbi cihaz
zh	医疗器械

en	Caution
ar	تحذير
bg	Внимание
cs	Upozornění
da	Forsigtig
de	Achtung
el	Προσοχή
es	Precaución
et	Ettevaatust!
fi	Huomio
fr	Attention
he	אזהרה
hr	Oprez
hu	Figyelem
it	Attenzione
lt	Atsargiai
lv	Uzmanību!
nl	Voorzichtig
no	Forsiktig
pl	Przeostroga
pt	Cuidado
ro	Atenție
ru	Осторожно!
sk	Upozornenie
sl	Svarilo
sv	Försiktighet
tr	Dikkat
zh	注意

REF

en	Catalogue number
ar	الرقم الكتالوجي
bg	Каталожен номер
cs	Katalogové číslo
da	Art.nr.
de	Artikelnummer
el	Αρ. καταλόγου
es	Número de catálogo
et	Tellimisnumber
fi	Tuotenumero
fr	Référence catalogue
he	מספר קטלוגי
hr	Kataloški broj
hu	Megrendelési szám
it	Riferimento di catalogo
lt	Užsakymo numeris
lv	Pasūtījuma numurs
nl	Artikelnummer
no	Artikkelnummer
pl	Numer katalogowy
pt	Número de catálogo
ro	Număr de catalog
ru	Номер по каталогу
sk	Katalogové číslo
sl	Kataloška številka
sv	Referensnummer
tr	Katalog numarası
zh	目录编号

LOT

en	Batch Code
ar	رمز الدفعة
bg	Код на партидата
cs	Číslo šarže
da	Lotnummer
de	Fertigungslosnummer
el	Κωδικός παρτίδας
es	Código de lote
et	Partii number
fi	Eränumero
fr	Numéro de lot
he	קוד מצווה
hr	Broj serije
hu	Gyártási tételszám
it	Codice di lotto
lt	Partijos kodas
lv	Partijas numurs
nl	Batch-code
no	Batchkode
pl	Numer partii
pt	Código do lote
ro	Codul lotului
ru	Код партии
sk	Číslo šarže
sl	Številka serije
sv	Lotnummer
tr	Parti Kodu
zh	批号

en Date of manufacture
ar تاريخ التصنيع
bg Дата на производство
cs Datum výroby
da Fremstillingsdato
de Herstellungsdatum
el Ημερομηνία κατασκευής
es Fecha de fabricación
et Tootmiskuupeev
fi Valmistuspäivämäärä
fr Date de fabrication
he תאריך ייצור
hr Datum proizvodnje
hu Gyártás ideje
it Data di fabbricazione
lt Pagaminimo data
lv Izgatavošanas datums
nl Productiedatum
no Produksjonsdato
pl Data produkcji
pt Data de fabrico
ro Data fabricației
ru Дата изготовления
sk Deň výroby
sl Datum izdelave
sv Tillverkningsdatum
tr Üretim tarihi
zh 制造日期

en Use by date
ar تاريخ الصلاحية
bg Срок на годност
cs Datum použitelnosti
da Udløbsdato
de Verwendbar bis
el Ημερομηνία λήξης
es Fecha de caducidad
et Kõlblik kuni
fi Viimeinen käyttöpäivämäärä
fr Date limite d'utilisation
he תאריך תפוגה
hr Upotrijebiti do
hu Felhasználható
it Data di scadenza
lt Tinka naudoti iki
lv Derīgs līdz
nl Uiterste gebruiksdatum
no Utløpsdato
pl Termin przydatności
pt Data de validade
ro Data de expirare
ru Использовать до
sk Použiteľné do
sl Uporabno do
sv Utgångsdatum
tr Son kullanım tarihi
zh 使用截止日期

GTIN

UDI

en Unique Device Identifier
ar المعرف المميز للجهاز
bg Уникален идентификатор на изделието
cs Jedinečný identifikátor prostředku
da Unik udstyrsidentifikation
de Produktidentifizierungsnummer
el Μοναδικό αναγνωριστικό προϊόντος
es Identificador único del dispositivo
et Unikaalne seadme identifikaator
fi Yksilöllinen laitetunniste
fr Identifiant unique des dispositifs médicaux
he מזהה מכשיר יחודי
hr Jedinstveni broj trgovačke jedinice
hu Egyedi eszközazonosító
it Identificativo univoco del dispositivo
lt Unikalus įrenginio identifikatorius
lv Unikāls ierīces identifikators
nl Unieke apparaatidentificatie
no Unik enhetsidentifikator
pl Unikalny identyfikator urządzenia
pt Identificação única do dispositivo
ro Identificator unic al dispozitivului
ru Уникальный идентификатор изделия
sk Jedinečný identifikátor zariadenia
sl Edinstveni identifikator pripomočka
sv Unik enhetsidentifikation
tr Benzersiz cihaz tanımlayıcı
zh 唯一设备标识符

en	Manufacturer
ar	الجهة المصنعة
bg	Производител
cs	Výrobce
da	Producent
de	Hersteller
el	Κατασκευαστής
es	Fabricante
et	Tootja
fi	Valmistaja
fr	Fabricant
he	יצרן
hr	Proizvođač
hu	Gyártó
it	Fabbricante
lt	Gamintojas
lv	Ražotājs
nl	Fabrikant
no	Produsent
pl	Producent
pt	Fabricante
ro	Producător
ru	Изготовитель
sk	Výrobca
sl	Izdelovalec
sv	Tillverkare
tr	Üretici Firma
zh	生产商

en	Keep dry
ar	احفظ المنتج جافاً
bg	Пазете от влага
cs	Uchovávejte v suchu
da	Opbevares tørt
de	Trocken aufbewahren
el	Διατηρείτε στεγνό
es	Mantener seco
et	Hoida kuivas kohas
fi	Pidettävä kuivana
fr	Conserver au sec
he	שמור יבש
hr	Čuvati na suhom mjestu
hu	Tartsa szárazon
it	Conservare in un luogo asciutto
lt	Laikyti sausoje vietoje
lv	Uzglabāt sausā vietā
nl	Droog bewaren
no	Oppbevares tørt
pl	Chronić przed wilgocią
pt	Conservar seco
ro	A se păstra uscat
ru	Беречь от влаги
sk	Uchovávať v suchu
sl	Hranite na suhem
sv	Förvaras torrt
tr	Kuru yerde tutun
zh	保持干燥

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English

Indications for Use

The YpsoPump® Orbit®soft Infusion Set is intended for the continuous subcutaneous delivery of insulin from the external infusion pump YpsoPump®.

Warnings

- ⚠ **Follow the instructions for use** of the YpsoPump® Orbit®soft Infusion Set and your YpsoPump®.
- ⚠ **Avoid infections.** Do not use the YpsoPump® Orbit®soft Infusion Set if the individual packaging is damaged, has already been opened, or if the cannula protector is missing. Do not use the YpsoPump® Orbit®soft Infusion Set after the expiration date on the packaging. If these instructions are not followed, there is a risk of infection. If you have any signs or symptoms of an infusion site infection, immediately replace your infusion set and contact your physician for treatment.
- ⚠ **Single use only!** Do not re-use, re-insert and/or re-sterilize the cannula. Re-use and/or re-insertion may cause damage to the cannula and lead to site irritation, infection and/or inaccurate insulin delivery. Re-sterilization may affect the integrity of the YpsoPump® Orbit®soft Infusion Set.
- ⚠ **Do not use the YpsoPump® Orbit®soft Infusion Set for longer than 72 hours.** Longer use may lead to site irritation, infection and/or inaccurate insulin delivery. Change your infusion site if it becomes itchy, red, inflamed or if you have an unexplained high blood sugar reading.
- ⚠ **Ensure correct delivery of insulin.** Before insertion, check that the tubing allows free flow of insulin, following the priming instructions of your YpsoPump®. This will prevent incorrect dosage caused by air in the fluid path or a clogged tubing. Use a new YpsoPump® Orbit®soft Infusion Set if priming is not successful. Never prime the infusion set tubing or attempt to free a blocked line while the tubing is connected to the inserted cannula base. You may accidentally deliver too much insulin. Do not re-insert the introducer needle into the soft cannula. Re-insertion may cause tearing or puncturing of the soft cannula, which may result in insulin leakage. Replace the YpsoPump® Orbit®soft Infusion Set if the tape becomes loose, if the soft cannula becomes kinked during insertion or if the cannula becomes fully or partially dislodged from the skin. Check frequently to make sure the soft cannula remains firmly in place. It must be completely inserted at all times to receive the full amount of insulin. 2–3 hours after insertion, a blood glucose test has to be performed to ensure adequate delivery of insulin. Therefore, do not change the YpsoPump® Orbit®soft Infusion Set immediately before bedtime.

Develop a conventional insulin injection or medication replacement plan with your healthcare professional if any part of the infusion system fails or system supplies are depleted.

If these instructions are not followed there is a risk of incorrect delivery of insulin. Seek immediate medical assistance if you have any health concerns.

⚠ Avoid bending or breaking the cannula/introducer needle.

A broken needle can remain stuck in the body or remain completely under the skin and may require surgical removal.

⚠ For Healthcare Professionals: Avoid recapping or destroying the introducer needle. Recapping or destroying the introducer needle can lead to accidental sharps injury which can transmit serious infections. Healthcare professionals must follow the specific directions of local policies for healthcare personnel when using infusion sets, including directions for recapping and disposing. Seek immediate medical assistance if you receive a sharps injury.

Caution (please note)

- Change the infusion site regularly in accordance with your healthcare professional's recommendations. If this is not done, you risk infusion site infection, insulin resistance and/or lipohypertrophy.
- Consult your healthcare professional to choose the correct cannula length. Your healthcare professional can provide recommendations regarding the specific type of infusion set that is best matched to your needs to ensure accurate delivery of insulin.
- Do not put perfumes, deodorants or insect repellents on the YpsoPump® Orbit®soft Infusion Set as this may affect the integrity of the infusion set. Protect the product from sunlight and heat. Store the YpsoPump® Orbit®soft Infusion Set in a dry location at room temperature and avoid high humidity.

Disposal

You must follow your healthcare professional's advice and local government regulations for disposal of all used needles/sharps.

English

Figure 1–2

Choose an infusion site away from the waistline, bones, scar tissue, belly button and recent infusion sites. Wash hands thoroughly and clean the infusion site according to the instructions provided by your healthcare professional. Prepare the infusion site by removing excess hair. We recommend to wipe the site in a circular outward motion, using a 70 % isopropyl alcohol pad, and let it air dry. Make sure the infusion site is completely dry before proceeding. The use of 70 % isopropyl alcohol supports the optimal adhesion of the tape. The use of other alcohol pads may result in premature tape failure and interruption of insulin delivery.

Figure 3–4

Attach the adapter of the infusion set straight onto the cartridge compartment of the YpsoPump® (see figure 3). Push the adapter gently down to pierce the septum of the insulin cartridge. Turn the adapter in a clockwise direction until you feel it slide into lock position. Check visually that the adapter is attached firmly and aligned with the pump case. After connection, make sure air bubbles are removed by priming the tubing assembly according to the priming instructions of the YpsoPump®.

Figure 5–6

Remove tape backing and be careful not to touch adhesive. Carefully remove the cannula protector from the cannula.

Figure 7–8

Stabilize the infusion site and insert the YpsoPump® Orbit®soft cannula with an angle of 90°. For insertion of the cannula, you may also use the Orbit®Insertor from Ypsomed. For maximum adhesion, press the tape to your skin and run your fingers over the adhesive for some seconds.

Figure 9–11

Hold tape against skin with two fingers of one hand. Gently squeeze tabs and remove the introducer needle carefully with the other hand. Dispose the introducer needle into a safe container unit immediately after injection to avoid accidentally pricking.

⚠ You may use the blue needle cover provided to cover the introducer needle after removal (not for healthcare professionals).

Figure 12–14

Attach the tubing cap straight to the cannula base. One to two clicks should be heard.

△ Rotate the tubing left and right, at least one full turn in each direction, while pulling upward on cap to ensure the tubing cap is fully engaged and the fluid path is opened.

After proper connection of the tubing to the cannula base, the newly inserted cannula has to be filled with insulin. Failure to do so results in missed insulin.

Figure A: Disconnection

To disconnect the tubing, gently squeeze tabs on both sides of the tubing cap and remove it. There is no need to cover the infusion base as the septum provides closure.

Before you reconnect the tubing cap remove the air bubbles from the tubing and swab septum with a 70 % isopropyl alcohol pad. Proper swabbing of septum reduces germ ingress into the fluid path. To reconnect, follow instructions in figures 12–14.

Monitor your blood glucose levels when disconnected and about 2–3 hours after reconnecting.

Priming volume for the tubing with adapter (volumes are approximate):

45 cm (18 inch): 10 units* (0.10 ml)

60 cm (24 inch): 12 units* (0.12 ml)

80 cm (31 inch): 15 units* (0.15 ml)

110 cm (43 inch): 18 units* (0.18 ml)

Priming volume for the cannula part (head) (volumes are approximate):

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- SV** Ypsomed AB
- UK** Ypsomed Ltd.
Ypsomed AG