CONTACTS

AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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Dear customer,
Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read the instructions for use carefully.
If you have any questions, contact your physician or medical retailer.

Intended purpose

GenuTrain OA is a medical device. It is an orthosis providing targeted relief and stabilization for more movement with osteoarthritis of the knee.

Indications

- Medial or lateral osteoarthritis of the knee
- Instability of the knee joint (mediolateral)
- Unicompartamental cartilage / bone lesion
- Condition after cartilage surgery and cartilage growth for necessary unilateral post-operative relief
- Following unilateral meniscal lesions
- Osteochondrosis dissecans (post-operative)
- Bone bruising (bone edema of the medial condylus of the femur / tibial plateau)

Risks of using this product

The benefits of this product are felt particularly during physical activity.

- Please remove your orthosis during longer periods of rest.
- Once you have been prescribed the GenuTrain OA, only use it according to your indication and any additional instructions given by a medical specialist². Please consult a specialist or your physician if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.

Contraindications

No clinically significant adverse reactions have been reported to date.

If you have any of the following conditions, the product should only be used after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Varicose veins (varicosis)

- Impaired sensation in the leg due to neurological abnormalities
- Circulatory disorders of the leg (e.g. in the case of diabetes mellitus)
- Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Putting on the GenuTrain OA

- To release the support straps, pull out the BOA® closure system ①.
- Take hold of the donning aids at the top on both sides, slip into the orthosis with an extended foot and pull it up and over the knee ②.
- Position the orthosis so that the kneecap is enclosed by the pad on the one side ③.

Setting the relief

- Press down on the BOA® closure system until it engages ④.
- Twist the BOA® closure system until you feel the desired relief ⑤.
After a few steps, please be aware of the following:
– The setting is too loose if the orthosis slips down or you cannot feel any relief or stabilization.
– The setting is too tight if you feel the support straps digging in. In this case, pull the BOA® closure system out again to release the system, press down until it engages, and twist it to the desired degree.
- Adjust the relief according to your individual activities.

Taking off the GenuTrain OA:

- To release the support straps, pull out the BOA® closure system ①.
- Take hold of the removal aids at the bottom on both sides and pull the GenuTrain OA down ④.
- Please note: If the movement of your GenuTrain OA was restricted by a specialist, undo the front part of both circular straps before putting on or taking off the orthosis each time ⑦.
- TIP: After undoing the straps, fold the Velcro tabs down again to avoid damaging the knitted fabric.

Cleaning instructions

- Please only wash the orthosis by hand at 30 °C / 86 °F (cold wash) using a mild detergent. All components can stay on the orthosis during washing. Leave the orthosis to air dry.
- Regular care guarantees optimum effectiveness of the product. Also observe the instructions on the label underneath the removal aid.
- Never expose the product to direct heat as this may impair its effectiveness. If you have any complaints, please contact your retailer. Only clean goods are handled.

General information for specialists

First fitting

- For medial relief, the flexible joint splint can be found on the outside of the knee ⑧.
- For lateral relief, the flexible joint splint can be found on the inside of the knee ⑧.

Adjust the splint anatomically, particularly for lateral relief. For stronger relief, you can adjust the splint to overcorrection. The joint splint made from flexible spring steel can be adjusted to the desired shape by hand ⑨.

Part of the body this product is used for

Knee joint

- An incorporated pad around the kneecap and various cushions

Assembly and fitting instructions

The product is to be adjusted by specialists.

Maintenance instructions

If handled and cared for correctly, the product is maintenance-free.

Technical specifications / parameters

Your product comprises:

- Knitted mesh
- BOA® closure system
- A lateral spring steel splint with a hinge
- Control elements made of plastic

Notes on reuse

This product is intended for your personal use only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain OA have not been observed.

Setting the restriction

- Open the handle of the hinge cover and turn it a quarter turn until the indicator points toward the unlock symbol ⑩. Remove the hinge cover.
- Restrict the orthosis to 0°, 30°, or 60° extension and 30°, 60°, or 90° flexion ⑥.
- Use the limiting pins provided according to the desired limiting angle and use the same setting at the top and bottom. The limiting pin grips are above the hinge bolt ⑦.
- Use the straps provided to increase stability and secure the limitation of movement. Both straps each have two Velcro tabs between two guide loops ③. Attach the first Velcro tab of both straps to the velour area directly below and above the hinge ④. Turn the GenuTrain OA round to the other side and fasten the second Velcro tab to the velour there ①.
- Thread the straps through the guide loops on the first Velcro area, secure both straps and tighten them if necessary ①.

TIP: Make sure that the Velcro tabs do not touch the knitted fabric.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the relevant authorities. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Thermoplastic elastomer (TPE), Silicone (SI), Stainless spring band steel, Spandex (EL), Ethylene Vinyl Acetate (EVA), Polyketone (PK), Polyoxymethylene (POM), Stainless steel, Cotton (CO), Polycarbonate (PC), Polyester (PES), Thermoplastic polyurethane (TPU), Polyethylene (PE)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

² A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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