
Cellacare® Gilchrist Easy

Schultergelenkorthese
Orthèse pour l'articulation de l'épaule
Shoulder joint orthosis
Órtesis para la articulación del hombro
Ortesi per l'articolazione della spalla
Orthese voor het schoudergewricht

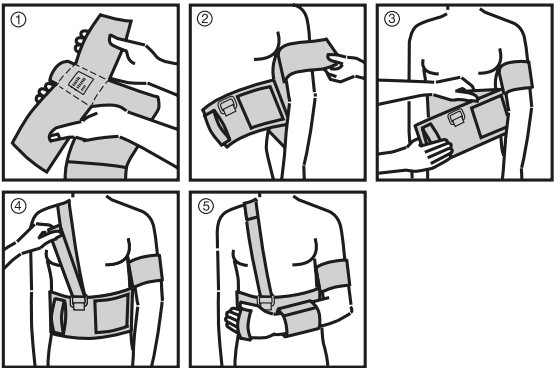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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your treating doctor or specialist shop.

Product description and performance characteristics:

Cellacare® Gilchrist Easy is an orthosis for immobilisation of the joints of the shoulder and the arm.

Product composition:

Cotton, polyamide, elastodiène (latex de caoutchouc naturel), polyester

Intended purpose:

Orthosis for immobilisation of the shoulder joints and the arm in a defined position

Indications:

- Acute painful muscle and/or tendon disorders of shoulder and upper arm
- Post-traumatic and/or post-operative immobilisation of shoulder and upper arm in case of:
 - Shoulder dislocation, luxation, distortion or contusion
 - Acute shoulder instability
 - Muscle and/or tendon injury
 - Surgical repair of tendons and ligaments in the area of shoulder and upper arm
 - Elbow injuries
- Upper limb paralysis

Contra-indications:

- Known allergy and/or hypersensitivity to any of the product components

Side effects:

In very rare cases, skin irritations and/or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Gilchrist Easy unless directed by a doctor. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The doctor will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Gilchrist Easy should only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and/or injuries in the application area (open, non-intact or injured skin) please contact your treating doctor/seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Gilchrist Easy is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.

- Cellacare® Gilchrist Easy must not be used beyond the last day of the month indicated on the packaging (expiry date).
-  This product contains natural rubber latex, which may cause allergic reactions.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Usage:

1. Open upper arm strap. Hold the strap's ends in both hands, so that you can read the sewn-in label (Fig. 1).
2. Fasten the upper arm strap approximately 2 cm above the bend of the elbow on the injured side (Fig 2).
3. Fasten the thorax strap (Fig 3).
4. Place the shoulder strap over the non-injured shoulder, pass the strap through the buckle and fasten the Velcro tape (Fig. 4).
5. Forearm and hand are held in place by applying the two remaining straps (Fig. 5).

Care recommendations:

Machine wash at 95 °C. May be dried in a dryer. Close the hook-and-loop fasteners and if possible use a mesh bag. Clean the hook-and-loop fasteners regularly so they retain the ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions:

Cellacare® Gilchrist Easy must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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