AUSTRIA
Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

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Material	
Base corsett, Front closure	Bridging frame – Spinova Unload Classic
Aluminium	Polycarbonate (PC)
Polyamide (PA)	Aluminium
Polyurethane (PUR)	EthyleneVinylAcetate (EVA)
Elastane (EL)	Polyoxymethylene (POM)
Polyester (PES)	Polyamide (PA)
Thermoplastic Polyurethane (TPU)	Polyurethane (PUR)
Polypropylene (PP)	Acrylate (AK)
Shell Spinova Immo Plus Classic	Back pad Spinova Support Plus Classic
Polyethylene, low density (PE- LD)	Polyamide (PA)
High-Grade Steel	Glass fiber (GF)
Polyamide (PA)	Polyester (PES)
Polyurethane (PUR)	Polyurethane (PUR)
Shell cover	Polypropylene (PP)
Polyester (PES)	Abdominal pad Spinova Support Plus Classic
Elastane (EL)	Spinova Immo Plus Classic
Polyamide (PA)	Polyethylene (PE)
	Ethylene Propylene Diene M-Class Rubber (EPDM)
	Polyurethane (PUR)
	Polyamide (PA)
	Cotton (CO)
	Polyester (PES)
	Elastane (EL)

30°C
86°F

Icons: Washing machine, No iron, No dry clean, No steam, No tumble dry

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en english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

The following information relates to these products:

Spinoa Stabi Classic

Spinoa Unload Classic

Spinoa Support Plus Classic

Spinoa Immo Plus Classic

Please also read the specific instructions relating to your product on the following pages.

Risks of using this product

⚠ Caution*

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

Your Spinoa spine orthosis must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application listed (parts of the body this product is used for). It is a prescribed product that should be applied under a physician's guidance. To ensure the best possible fit, the orthosis must be individually adjusted by a trained specialist¹. Please refer to the care instructions on the sewn-in label. The sewn-in label contains information on the size, manufacturer, care instructions, and CE marking and is located on the inside of the base support.

Use in combination with other products, e.g. as part of compression treatment (compression body support, lymph compression stockings), must be discussed beforehand with your physician. Should you notice any unusual changes or increased symptoms, please contact your physician without delay.

Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.

No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids. No side effects that affect the

entire body have been reported to date. Correct usage / fitting is assumed. Do not put the orthosis on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur.

If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

After surgery, make sure that the wound is regularly checked by a physician.

Note: Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the orthosis.

Application instructions

Putting on the orthosis

The Spinoa spine orthosis must be adapted by a trained specialist¹ before the first fitting.

1 Position the orthosis centrally, at waist height, so that the lower back is covered. The stamped arrows on the inside of the abdominal fastening should be pointing upward. **2** Slide your fingers from the side into the finger pockets provided on the fastening and pull the pockets evenly to the side. Now pull the fastenings forward. **3** Lift the abdomen slightly from below with the left half of the fastening and position the right half of the fastening on top of the left half so that it can be fastened to it. Now take hold of the tensioning straps with both hands. **4** Pull these simultaneously and evenly, to the side at first, until the desired degree of stabilization is achieved. **5** Fasten the tensioning straps to the abdominal fastening.

Taking off the orthosis

To take off the orthosis, start by undoing the tensioning straps. Then reach into the pockets of the abdominal fastening and unfasten the closure. After removing the orthosis, refasten the abdominal fastening to avoid damage.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries to the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
- Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the product have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical device, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2020-02

Spinova Stabi Classic

Intended use / part of the body this product is used for

Spinova Stabi Classic is a medical device. It is an orthosis for moderately stabilizing the lumbar spine using corset stays.

Indications

- Moderate (pseudo) radicular lumbar syndrome
- Moderate lumbosciatica with slight deficiencies in the case of intervertebral disk protrusions / herniated disks
- Condition after Herniated disks (non-surgical / post-operative)
- Condition after Discectomy
- Spondylolisthesis, grade I

Components

- A – Base support
- B – Abdominal fastening with finger pockets
- C – Tensioning straps
- D – Aluminum stays

Assembly and fitting instructions

The Spinova Stabi Classic is supplied as a pre-assembled orthosis in standard sizes. The aluminum and corset stays, the abdominal fastening and the tensioning straps should be individually adjusted by a trained specialist.

Cleaning instructions

If necessary, you can wash the orthosis by hand with a mild cleaning agent. Leave the orthosis to air dry.

Technical specifications / parameters

The Spinova Stabi Classic consists of a base support with aluminum and corset stays, an abdominal fastening, and two tensioning straps.

Spinova Unload Classic

Intended use / part of the body this product is used for

Spinova Unload Classic is a medical device. It is an orthosis for moderately reducing and relieving lordosis of the lumbar spine with removable bridging frame.

Indications

- Stable vertebral fractures (lumbar spine) with vertebral body damage, relieving the rear edges of vertebrae
- Spondylolysis / spondylolisthesis, grades I–II
- Moderate lumbosciatica with muscular deficiencies in cases of significant intervertebral disk protrusions / herniated disks
- Lumbar spinal canal stenosis, also with paresis (non-surgical, post-operative)
- Moderate (pseudo) radicular lumbar syndrome
- Facet syndrome
- Condition after Discectomy

Components

- A – Base support
- B – Abdominal fastening with finger pockets
- C – Tensioning straps
- D – Aluminum stays
- E – Bridging frame

Assembly and fitting instructions

The Spinova Unload Classic is supplied as a pre-assembled orthosis in standard sizes. The aluminum and corset stays, the abdominal fastening, the bridging frame, and the tensioning straps must be individually adjusted by a trained specialist.

Cleaning instructions

If necessary, you can wash the Spinova Unload Classic by hand with a mild cleaning agent. Unfasten the bridging frame **E** before washing. Refasten the bridging frame after drying. To do this, place the orthosis

in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Fasten the bridging frame firmly around the struts of the frame using the Velcro tabs (the arrows on the bars should be pointing upward). Secure both bars on the bridging frame. Push one Velcro strip through the eye of the bar from the front **III**.

Technical specifications / parameters

The Spinova Unload Classic is made up of a base support with aluminum and corset stays, an abdominal fastening, two tensioning straps, and a bridging frame.

Spinova Support Plus Classic

Intended use / part of the body this product is used for

Spinova Support Plus Classic is a medical device. It is an orthosis for moderately stabilizing the lumbar spine with an anatomically contoured back pad.

Indications

- Vertebral fractures (lumbar spine) with vertebral body damage – relief of the anterior vertebral edges
- Spondylolysis / spondylolisthesis, grade II
- Moderate lumbosciatica with muscular deficiencies in cases of significant intervertebral disk protrusions / herniated disks
- Lumbar spinal canal stenosis, also with paresis (non-surgical, post-operative)
- Moderate (pseudo) radicular lumbar syndrome
- Severe degeneration / muscular insufficiency
- Condition following herniation (post-operative / non-surgical)
- Condition following discectomy
- Condition following spinal fusion / kyphoplasty (one or more levels)
- Osteoporosis

Components

- A – Base support
- B – Abdominal fastening with finger pockets
- C – Tensioning straps
- D – Aluminum stays
- F – Large back pad
- G – Abdominal pad

Assembly and fitting instructions

The Spinova Support Plus Classic is supplied as a pre-assembled orthosis in standard sizes. The aluminum and corset stays, the abdominal fastening, and the tensioning straps must be individually adjusted by a trained specialist. The bridging pad and abdominal pad must be fastened to the base support.

Cleaning instructions

If necessary, you can wash the Spinova Support Plus Classic by hand with a mild cleaning agent. Unfasten both pads **F/G** before washing. Refasten both pads after drying. To do this, place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Fasten the back pad firmly onto the center of the support with the tapered end pointing downward. Now attach the abdominal pad to the right abdominal fastening **IV**.

Technical specifications / parameters

The Spinova Support Plus Classic is made up of a base support with aluminum and corset stays, an abdominal fastening, two tensioning straps, a back pad, and an abdominal pad.

Spinova Immo Plus Classic

Intended use / part of the body this product is used for

Spinova Immo Plus Classic is a medical device. It is an orthosis for immobilizing the lumbar spine with mobilization function.

Its special design permits controlled treatment according to an individually adaptable multi-stage regimen.

Stage 1 (Stabilization phase):

To secure the spine after surgery or at the start of non-surgical treatment, use the orthosis in combination with the shell.

Stage 2 (Mobilization phase):

During the initial stages of mobilization with a progressive recovery process, the shell is removed and the base support is reinforced with the aluminum and corset stays in the affected lumbar / sacral area.

Please ask your physician when you can remove the shell.

Indications

- Stable vertebral fractures (lumbar spine) with vertebral body damage, relieving the front and / or rear edges of vertebrae
- Most severe lumbosciatica with muscular deficiencies in cases of intervertebral disk protrusions / herniated disks
- Lumbar spinal canal stenosis, also with paresis (non-surgical, post-operative)
- Tumors (metastases)
- Severe degeneration / muscular insufficiency of the spine
- Spondylolysis / spondylolisthesis, grades II–III
- Facet syndrome / spondylitis
- Most severe (pseudo) radicular lumbar syndrome
- Foraminal stenosis
- Condition after herniated disks (non-surgical / post-operative)
- Condition after hiscectomy
- Condition after spondylodesis / kyphoplasty (one or more levels)
- Osteochondrosis

Components

- A – Base support
- B – Abdominal fastening with finger pockets
- C – Tensioning straps
- D – Aluminum stays
- G – Abdominal pad
- H – Shell with fabric top cover

Assembly and fitting instructions

The Spinova Immo Plus Classic is supplied as a pre-assembled orthosis in standard sizes. The shell, the stays, the abdominal fastening, and the tensioning straps must be individually adjusted by a trained specialist. The abdominal pad must be fastened to the base support.

V To detach the fabric top cover, slip both the rubber bands on the base support over the front edges of the shell. Then remove the fabric top cover. To detach the shell from the orthosis, undo the middle screw in the back area. Remove the shell from the base support, then you can use the base support without it.

Cleaning instructions

Wipe the shell with a soft cloth and a mild cleaning agent. Then leave the orthosis to air dry. When using the Spinova Immo Plus Classic without the shell (mobilization phase), the orthosis can be washed by hand if necessary.

Technical specifications / parameters

The Spinova Immo Plus Classic is made up of a base support with aluminum and corset stays, an abdominal fastening, two tensioning straps, a shell, and an abdominal pad.

Specialists¹

The following information relates to these products:

Spinova Stabi Classic

Spinova Unload Classic

Spinova Support Plus Classic

Spinova Immo Plus Classic

Please also read the specific instructions relating to each product.

General instructions

1 – 5 Adapt the orthosis according to the areas of application (indications) and the anatomy of the patient. Check the optimal and individual fit of the orthosis after the first fitting. Practice the correct procedure for putting on the orthosis with the patient. No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby canceling any product liability. The Spinova spine orthosis is intended for the treatment of one patient only.

Position the orthosis centrally, at waist height, so that the lower back is covered. The stamped arrows on the inside of the abdominal fastening should be pointing upward. To ensure optimal effectiveness, the orthosis should be positioned as low as possible without pressing against the groin area while the patient is sitting upright.

Fitting / assembly

Fitting the stays (dorsal aluminum stays and corset stays)

To ensure that the orthosis fits optimally without pressure points, the stays can be adjusted to suit the patient's anatomy **D**. Take the stays out of the stay pockets on the base support and shape them to the patient's individual body shape. Reinsert the stays into the stay pockets.

Shortening the tensioning straps **I**

First loosen the Velcro ends of the tensioning straps and attach them onto the middle of the fastening. Either you or your patient must then take hold of the tensioning straps in both hands and pull them forward until the desired level of tension is achieved. Fasten the tensioning straps with the Velcro ends at the optimum length and shorten the strap accordingly.

Adapting the abdominal fastenings to the patient's body shape

You can adjust the angle and width of the abdominal fastenings to suit individual requirements. To do so, loosen the abdominal fastenings and fasten them in a new position. Any protruding velor fabric on the base support can be cut off with fabric scissors once the product has been fitted. Protruding Velcro must be covered with a velor strip to avoid damage. In principle, you can also cut off the outer Velcro strip completely if it is not required. **II**

Spinova Stabi Classic

Fitting / assembly

Adjust the stays, tensioning straps, and abdominal fastening as described above.

Spinova Unload Classic

Fitting / assembly

Adjust the stays, tensioning straps, and abdominal fastening as described above.

Adjusting the bridging frame

To adjust the bridging frame, unfasten it from the base support. Remove the two Velcro strips from the bridging frame and open the two Velcro tabs on the base support. The aluminum stays on the bridging frame can be individually shaped. Then reattach the bridging frame, positioning it according to the patient's body shape so that its delordosing function works to optimum effect. The arrows on the bars of the frame and the inside of the abdominal fastenings should be pointing upward. Please note that the Spinova Unload Classic is available in two heights **III**.

Spinova Support Plus Classic

Fitting / assembly

Adjust the stays, tensioning straps, and abdominal fastening as described above.

Positioning the pads

Place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward **IV**.

Fasten the large back pad firmly onto the center of the support with the tapered end pointing downward. Now attach the abdominal pad to the left abdominal fastening. The upper edge of the back pad should sit approximately 3 cm below the shoulder blades. Please note that the Spinova Support Plus Classic is available in two heights.

Spinova Immo Plus Classic

Fitting / assembly

Adjust the stays, tensioning straps, and abdominal fastening as described above.

Tools required

2.5 mm Allen key (available as an accessory)

Removing the shell from the orthosis

V Undo the middle screw on the back. Slip both the rubber bands on the base support over the front edges of the shell. Remove the base support from the back shell. If necessary, remove the fabric top cover from the shell.

Setting the height, width, and angle of the shell

After removing the shell from the base support, loosen the nut in the center of the shell by hand and undo the upper and lower screws with an Allen key so that both halves of the shell can be moved easily toward or away from each other. Position the shell on the patient's torso so that the recesses of the pelvic frame sit directly above the iliac crests. Shift and bend the two halves of the shell until it is as flat as possible against the patient's torso. Then tighten the three screw connections up again, making sure that they are aligned as symmetrically as possible **V**. The upper edge should end approximately 3 cm below the shoulder blades. If necessary, the upper edge can be shortened to change the height of the shell. Please note that the Spinova Immo Plus Classic is available in

two heights. To attach the shell to the base support, carry out the steps listed in the **Removing the shell** section in reverse.

Note: The plastic can be cut with standard leather shears. Smooth the cut edge (e.g. with sandpaper) to avoid causing injuries.

Positioning the abdominal pads

Place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Attach the abdominal pad to the left abdominal fastening.

^{*} Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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© 2024 Spinova AB. Alla rättigheter förbehållna.