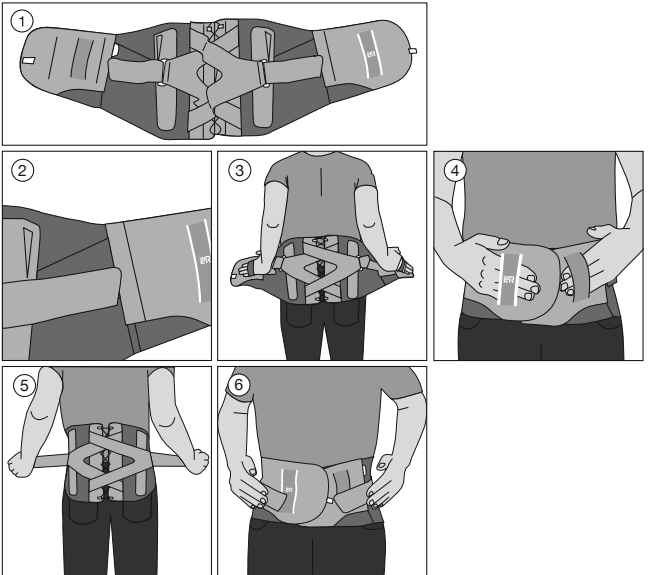
de

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Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:
Cellacare® Dotop Comfort is a back orthosis with a relieving and stabilising effect. Four integrated adjustable steel stays and an additional 3-point strap system contribute to the relief and support of the lumbar spine.

Product composition:
Polyamide, elastane, polyester, polyurethane, polyacetal, steel, polypropylene, polyolefins, epoxy-polyester, polyethylene, cotton

Intended purpose:

Orthosis for relief and stabilisation of the lumbar spine

Indications:

- Acute or chronic nonspecific lumbar spine pain syndrome caused by incorrect loading (abnormal biomechanical stress) or stress (in terms of load) of muscles and ligaments of the lumbar spine and sacrum area
- Acute or chronic specific lumbar spine pain syndrome in the case of
 - Osteochondrosis in the lumbar spine area
 - Intervertebral disk protrusions / prolapses in the lumbar spine
 - Spondylarthritis in the lumbar spine area
 - Spondylolysis with or without spondylolisthesis (grade I)
- Postoperative care of intervertebral disc surgery

Contra-indications:

- Known allergy and / or hypersensitivity to any of the product components
- Severe back injuries / disorders such as instability / fragility of the thoracic and / or lumbar spine with vertebral fractures, in case of bone tumours or metastases or advanced osteoporosis

Side effects:

- In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Dotop Comfort unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.

- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Dotop Comfort shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of pregnancy, the patient's treating physician needs to give the approval before using the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Dotop Comfort must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Dotop Comfort is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Dotop Comfort must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Care recommendations:
Wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Dotop Comfort to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closure and if possible use a mesh bag. Clean the hook-and-loop closure regularly so it retains its ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions:
Cellacare® Dotop Comfort must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:
In case of a serious incident, contact the manufacturer and the responsible health authorities.

Usage information:

Preparation:

- Apply Cellacare® Dotop Comfort while standing.
- For hygienic reasons wearing a shirt underneath the orthosis is recommended.
- Open the hook-and-loop fastener and flatten out the orthosis in front of you (Fig. 1). Make sure that the ends of the additional straps are attached to the straps (Fig. 2).

Usage:

- Apply the orthosis around the waist. Make use of the grips (Fig. 3).

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