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
INSTRUCTION FOR USE

DESCRIPTION OF PARTS

Please read carefully before using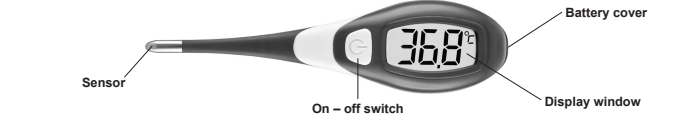


SPECIFICATIONS	
Range:	32,0 °C to 43,9 °C (89,6 °F to 111,1 °F) Below 32,0 °C (89,6 °F) display „L“ (°C/°F) Above 43,9 °C (111,1 °C) display „H“ (°C/°F) Measurement of body temperature 0,1 °C (°F) ± 0,1 °C (34,0 °C to 42,0 °C); ± 0,2 °C (other range) Accuracy: ± 0,2 °F (93,2 °F to 107,6 °F); ± 0,4 °F (other range)
Intended use:	Liquid crystal display 3 1/2 digits CR1225 (3 V)
Resolution:	0,4 mW in measurement mode
Accuracy:	More than 160 hours of continuous operation
Display:	IP22 (effective against solid foreign objects with a finger; protected against dripping water)
Battery (included):	117 mm x 30 mm x 16 mm Approx. 20 grams including battery
Power consumption:	Approx. 10 second sound signal when no relevant increase in temperature is detected
Battery life:	Built-in memory enabling display of the previously measured value
IP classification:	Temperature: 10 °C to 40 °C (50 °F to 104 °F) Humidity: 15 % to 95 % R.H.
Dimensions:	It is only possible to measure temperatures that are above the ambient tem-perature.
Weight:	Temperature: -25 °C to +55 °C (-13 °F to 131 °F) Humidity: 15 % to 95 % R.H.
Beeper:	The thermometers meet the requirements of EN 12470-3 (Electrical Compact Ther-mometer, maximum), with the exception of 6.3.7 (water resistance).
Memory:	Geratherm® is certified in accordance with Council Directive 93/42/EEG and EN ISO 13485 and is entitled to affix the CE-mark CE0197 (Notified Body: TÜV Rheinland LGA Products GmbH).
Working Conditions:	The present medical product is a device with extremely high degree of noise immu-nity. Therefore, the enclosure with the EMC tables was not printed out.
Storage Conditions:	
Guarantee of Quality:	
Electromagnetic compatibility (EMC):	

PRECAUTIONS

- For best results, do not use the thermometer for Axillary measurement. Use it only for oral or rectal measurement.
- Use of this instrument is not intended as a substitute for consultation with your physician. Prolonged high fever requires medical attention especially for your children. Please contact your physician.
- Child's temperature shall be taken under adult's surveillance.
- Keep out of the reach of unsupervised children. Choking resulting from a child swallowing a small part that has become detached from the thermometer.
- Do not boil, bite, bend, drop or disassemble this thermometer.
- Do not attempt to disassemble the unit except to replace the battery.
- Avoid taking temperature in 30 minutes after physical exercises, bathing or dining.
- Do not store, use and transport the unit under direct sunlight, at a high temperature, or in high humidity or dust. Performance may be degraded.
- Stop using the thermometer if it operates erratically or if the display malfunctions.
- If the thermometer has been stored at below-freezing temperatures, allow the thermometer to warm naturally to room temperature before using.


BATTERY REPLACEMENT

- When the symbol „“ appears, the battery needs to be replaced.
- Put the battery cover and the plug off in the direction as shown.
- Use a toothpick or insulated probe to remove the battery. Avoid using any sharp metal ob-ject in this operation.
- Place a new battery type with positive side up and the negative side facing down.
- Turn the battery module and put it back into the thermometer.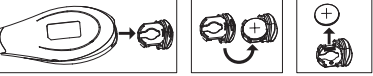

Discard old battery carefully, out of reach of young children. Do not dispose of the battery in a fire. It may explode.

HOW TO USE

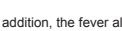- Clean or disinfect the probe before every measurement with Ethyl.
- Press the ON/OFF button. The display will show  or , in addition, the fever alarm will ap-pear in green, yellow and red.
- Release the power switch and the display will show „L“ (°C/°F) with „C“ (°F) flashing.
- Position the probe (oral, rectal, axillary etc.).
- As soon as the thermometer does not register any further relevant increase in temperature, „C“ (°F) stops flashing and the maximum value measured up to that time is shown, accompanied by a series of audible signals. The indicator lights up in the corresponding color of the fever alarm. (green, yellow, red).
- The unit will automatically turn off in 30 seconds (approx.). However, to prolong battery life, it is best to turn the thermometer off by pressing the ON/OFF button once the temperature has been noted.

Temperatur (°C)	Temperatur(°F)	Fever alarm indicator
35,9 ± T	96,7 ± T	clear
36,0 ± T + 37,1	96,8 ± T + 98,8	green
37,2 ± T + 37,4	98,9 ± T + 99,4	yellow
37,5 ± T + 43,9	99,5 ± T + 111,1	red
43,9 ± T	111,2 ± T	clear

SPECIAL FEATURES

Normal Temperature Alarm: An alarm will sound when the maximum temperature has registered and is ready-to-read. The „Normal Temperature“ alarm (four quick beeps followed by a pause) will sound repeatedly for approxi-mately 5 seconds.

Last Memory Recall: If the ON/OFF button is pressed to activate the thermometer, hold the button for 2 seconds. The display 188,8 changes to show the last measured temperature. This reading is displayed as long as the button is held down. When released, the thermometer is in "ready-to-measure" mode and the recalled temperature will be deleted.

Fever Alarm: If the temperature registers higher than 37,5 °C (99,5 °F) the fever alarm (a quick beeping sound which occurs continuously for about 5 seconds) will sound when the peak temperature has registered. Also, the temperature will keep flashing.

Switch between Celsius and Fahrenheit: After turning on the thermometer, the Centigrade or Fahrenheit mode can be switched within the first 5 seconds by pressing and holding the ON/OFF button for 2 seconds.

NOTE: You may put a sanitary plastic probe shield on the sensor tip to help prevent cross-infection and recon-tamination. Use of the probe cover may result in a 0,1 °C (0,2 °F) difference from actual temperature. Dispose of any used probe shield and store the unit in a safe place.

MESSMETHODEN

It is only possible to measure temperatures that are above the ambient temperature.

ORAL USE
Place the probe well under the patient's tongue. Instruct the patient to keep their mouth closed .
A normal temperature by this method is usually considered between 36,0 °C and 37,5 °C (96,8 °F and 99,5 °F).

RECTAL USE
If rectal temperature is recommended by your doctor, you may use a probe cover. Insert thermometer into the probe cover. Lubricate the probe cover with a water soluble jelly for easier insertion.
Do not use petroleum jelly. Insert tip of probe no more than 1,5 cm into rectum.
STOP if you meet any resistance. In general, the rectal temperature is around 0,5 °C to 1,0 °C (1,0 °F to 2,0 °F) higher than oral temperature.

AXILLARY USE
Wipe armpit with a dry towel. Place probe in the patient's armpit and keep the patient's arm pressed firmly against their side.
In general, the axillary temperature is around 0,5 °C to 1,0 °C (1,0 °F to 2,0 °F) lower than oral temperature.

If you compare temperatures between two thermometers, it is possible to have two different readings even if both thermometers are within their accuracy tolerances. Differences in how you take each reading and the thermom-eter's tolerance range can account for nearly 0,1 °C (0,2 °F) variability.

REMEMBER: Find your normal temperature and watch for changes.

CLEANING AND DISINFECTION

For household use, always clean the probe head before and after each use.
Wash the tip with a solution of mild detergent and cool water.
Disinfect the thermometer by wiping the sensor and lower stem with a cloth dipped in a household antiseptic solution such as rubbing alcohol (recommendation: 70 % isopropyl alcohol).
Do not boil or clean in autoclave.

DISPOSAL

Observe the applicable regulations when disposing of the thermometer and batteries. This thermometer must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regard-less of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.
Please remove the batteries before disposing of the thermometer.
Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

LIMITED WARRANTY

This thermometer is guaranteed for two years from the date of purchase against manufacturer's defect under normal use. If your unit does not function properly due to defective parts or assembly we will repair it free of charge. All parts are covered by this warranty excluding the battery. The warranty does not cover damage to your unit due to improper handling. We recommend that the accuracy of the unit be checked every 2 years by an authorized laboratory.

SYMBOL INDEX

	Follow instructions for use		Type BF applied part
	Keep dry		Manufacturer
	Store between 15 % and 95 % R.H.		Caution! Read the instruc-tion manual.
	Store between -25 °C and +55 °C		Batch code (mm/yyyy; month/ year)
	The device must not be dis-posed of with household waste.		Geratherm Medical AG Fahrenheitstrasse 1 98716 Geschwenda Germany www.geratherm.com

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