Omnitest® 3 Control

GB Control solution

CE 0123

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GB Instructions for use

IMPORTANT

Please read this information and your Omnitest® 3 meter and test strip manual before using Omnitest® 3 Control solution.

Intended use:

Only for use with Omnitest® 3 meter and Omnitest® 3 test strips as a quality control measure to verify the accuracy of the blood glucose test results. The Omnitest® 3 Control solution is for in vitro diagnostic use.

WARNING

Keep the control solution away from children. A child could choke on the cap or the bottle. The control solution contains agents that may be harmful if swallowed or applied to the skin or eyes. Do not drink the Omnitest® 3 Control solution, it is meant to be used on the test strips only.

Summary and explanation:

The Omnitest® 3 Control solution is used to check that the blood glucose monitoring system consisting of the meter and the test strips is working correctly. The Omnitest® 3 Control solution is a liquid containing a measured amount of glucose which reacts with the Omnitest® 3 test strip. If used according to the same procedure as for blood, the control solution should give results within the expected range printed on the test strip vial. Obtaining a single result within the expected range is sufficient to verify the system performance. Use only Omnitest® 3 Control solution for the Omnitest® 3 monitoring system.

The control solution should be used, e.g.

- Whenever you suspect the meter or test strip is not functioning properly.
- If your blood glucose test results are not consistent with your symptoms or if you think they are not accurate.
- If you have dropped the meter.
- For quality control at the point of care usage.
- For teaching or learning the system.

Storage and handling:

- Store the Omnitest® 3 Control solution in a dry place between 2 – 30 °C (36 – 86 °F). Keep out of direct sunlight.
- Do not freeze. Do not use control solution which had been stored under inappropriate conditions.
- Store control solution in its original vial only. Do not fill the control solution in new vials or in any other container.
- Wipe off the dispenser tip of excess solution with a clean tissue before replacing the cap.
- Do not use control solution after the expiration date printed on the package or vial since it may cause inaccurate results.
- Make a notation of the date on the vial label when you first open it. Discard remaining Omnitest® 3 Control solution 3 months after first opening a bottle.

Using control solution:

To perform a control solution test, follow the same test procedure as for blood glucose test, substituting control solution for the drop of the blood. Refer to the Omnitest® 3 meter manual for further test instructions (see "Testing Your Blood Glucose"). If the control solution is cold, do not use until the solution has warmed up to room temperature.

1 Insertion of test strip

Control solution tests have to be done in a temperature range 20–25 °C (68–77 °F). Insert a test strip. Push the test strip until it will go no further without bending the strip. The meter will automatically turn on.

2 Apply control solution

Shake the control solution vial gently. Remove the cap. Squeeze the vial, discard the first drop and wipe off the dispenser tip with a clean tissue to ensure an accurate result. If large bubbles are present at the tip of the vial, wipe the bubbles off before applying the drop of control solution to the test strip. Squeeze the vial again to get a drop. Apply a drop of control solution on a clean and non-absorbing base. Guide the tip of the test strip toward the drop until it is drawn up from the test strip. Once you hear the "beep", it means the test strip is completely filled. The meter begins to count down 3 to 1 and then the result is displayed.

3 Comparing control solution results

The actual lot-specific ranges are printed on the Omnitest® 3 test strips vial (Control M ...; Control H ...). The Omnitest® 3 Control solution test should fall within the range stated in mg/dL or mmol/L. The average value of the range is printed underneath (mean). If test results fall outside the range, repeat the test. Out-of-range results may be caused by one or more of the following causes:

- Error in performing the test.
- Failure to shake the control solution vial gently.
- Expired or contaminated control solution.

- Control solution that is too warm or too cold.
- Failure to discard the first drop of control solution and wipe the dispenser tip clean.
- Contamination of the control solution e.g. if the drop of the control solution is taken back into the control vial.
- Meter malfunction.
- Test strip deterioration.

IMPORTANT

Do not use the Omnitest® 3 system to test your blood glucose until you get a control solution test result that falls within the range.

4 Marking function

Press **▲** or **▼** button after the measurement to choose the control solution symbol **▲** among the available symbols.

NOTE

Mark all control solution tests with symbol **▲** to distinguish from blood glucose tests in the meter memory. Marked control solution test results will not be included in your averages.

Chemical composition:

The Omnitest® 3 Control solution is an aqueous solution with D-glucose as the reactive ingredient.

Ingredients and approximate concentrations of the Omnitest® 3 Control solutions:

Control M	Control H
0.11 wt % glucose	0.26 wt % glucose
99.89 wt % nonreactive ingredients	99.74 wt % nonreactive ingredients

If you have any questions about use of the Omnitest® 3 product, please contact your nearest B.Braun representative or go to www.omnitest.bbraun.com

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