AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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These instructions for use can also be downloaded from:
www.bauerfeind.com/downloads

Dear Customer,

Thank you for choosing a Bauerfeind product. Please read the **instructions for use** carefully and contact your physician or medical retailer if you have any questions.

Intended use

The LumboTrain / LumboTrain Lady is an active support for the back muscles for discomfort in the lumbar spine region. Positioned correctly, the massage pad together with the compression knit wrapped around the body provide an intermittent compression massage. This relieves the affected area and reduces pain. It activates the muscles and improves stability.

Indications

- Non-specific low back pain (acute lumbar spine syndrome)
- Weakness and dysfunction of the torso-supporting muscles (slight degeneration / muscular insufficiency of the spine)
- Intervertebral disk surgery (discectomy)
- Conservative treatment following an intervertebral disk prolapse (conservative after prolapse)

Risks of using this product

- The use and duration of use must be discussed with the physician who prescribed the LumboTrain. He will also inform you whether the LumboTrain can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the LumboTrain as well as the instructions for use. The product may be damaged if it is not used correctly or is altered arbitrarily and the warranty will not be valid.
- The effects of this support are felt primarily during physical activity. Take the support off during extended rest phases – e.g. sleeping and long periods of sitting down.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience unusual changes connected to the use of the LumboTrain – e.g. an increase in discomfort – stop using it and contact a physician immediately.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, the product should only be used after consultation with your physician:

1. Skin disorders / injuries in the affected part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
2. Impaired sensation
3. Severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion)

Application instructions

Wear the support over a T-shirt or a vest.

Applying the pad

Fasten the pad to the fleece band in the back section of the support, taking the anatomical contouring into account, so that the opening for the silicone dots runs vertically down the center.

Applying the LumboTrain / LumboTrain Lady

1. Wrap the support around the body so that the pad  is positioned in the middle of the back.
2. Place your fingers in the finger pockets provided on the right and left of the fastening and pull the support forward evenly .
3. Press the left half of the fastening onto the abdomen and pull the right half until it can be fixed in place with a Velcro fastening .
4. When fastening the support, remove your fingers from the left hand strap first and then from the right hand strap .

Cleaning instructions

Note: Never expose the support to direct heat (e.g. heaters, sunlight, leaving in your car, etc.) This can impair the effectiveness of the support.

Please refer to the care instructions on the sewn-in label. This can be found on the inner part of the knitted fabric.

Please note the following:

1. Remove the back pad before washing. Wash the pad by hand.
 2. Close the fastening of the support.
 3. We recommend using a lingerie bag.
 4. The support is not suitable for tumble drying.
- Wash your support regularly to retain the compression of the knitted fabric.

To make a complaint, please contact the retailer that provided you with the product.

Note: Only clean goods are handled

Notes on reuse

This product is intended for your treatment only. Do not pass the support on to other people.

Warranty

The statutory warranty regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. It is your first local point of contact before the manufacturer.

Please use the LumboTrain in line with expert instructions and the enclosed instructions for use. Take note of the care instructions and do not make any alterations to the product. Only aids that function perfectly will help you. Failure to follow the instructions may also compromise the warranty.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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